quattro[®]

Leading in Orthodontic Implants

psm-medical.com

Quality is never an accident.

It is always the result of high intention, sincere effort, intelligent direction and skillful execution; it represents the wise choice of many alternatives.

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Introduction | Einführung

Orthodontic Mini-Anchor System

The use of Mini-Anchor Screws or Temporary Anchorage Devices (TADs) has become a standard of care within the orthodontic community over the past few years. First offered in 2000 the **quattro**® TAD design has continuously been changed and improved to become the **quattro**® TAD family of today. The **quattro**® TADs even after more than a decade of production are still unmatched in terms of versatility, quality and maximal success rate. With the **quattro**®.mini (see separate catalogue) screw a new even smaller bracket head design is launched in 2012 offering a new dimension in patient comfort based on the outstanding **quattro**® design. Since year 2000 close to 100.000 **quattro**® TADs have been placed in more than 70 countries worldwide.

The **quattro**® System is ideally enhanced with the unique **BENEFit**®/**BENEplate** System (developed in cooperation with University of Düsseldorf, Germany) offering exchangeable abutments and connecting plates. This opens up new possibilities in palatal based appliances popularly used for molar distalization (see below QR Video and page 15). **quattro**® and **BENEFit**® a complete TAD System using the same basic instrumentation keeping inventory manageable and very cost effective.

Features:

- Clinically proven applications
- Biocompatible, high-strength Titanium
- Self-drilling & self-tapping technique
- Reliable & secure anchorage
- Innovative screwhead designs
- Stronger conical body design
- Deeper threads & denser pitch design
- Available in multiple sizes

Benefits:

- Provides improved higher strength
- Immediate load-bearing of heavy forces
- Easy to combine with applications of orthodontic accessories
- Better mechanical retention
- Avoids gingival impingement
- Reduced treatment time
- Easy to insert and remove
- Variable treatment spectrum

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PSM – The Original **quattro**® TAD System

The *quattro*® Screw Family | Die *quattro*® Schrauben-Familie

quattro®standard

quattro®hook

quattro®plus

quattro®plus V

One Screw – Unlimited Possibilities

The *quattro*® TAD system offers 4 different screwhead designs which are perfectly adapted to the Orthodontist's appliances used daily. All *quattro*® screws (1.5 mm to 2.3 mm) feature the identical and well-proven thread design for easy insertion and most secure and satisfactory treatment results.

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Atraumatic platform design reduces soft tissue pressure.

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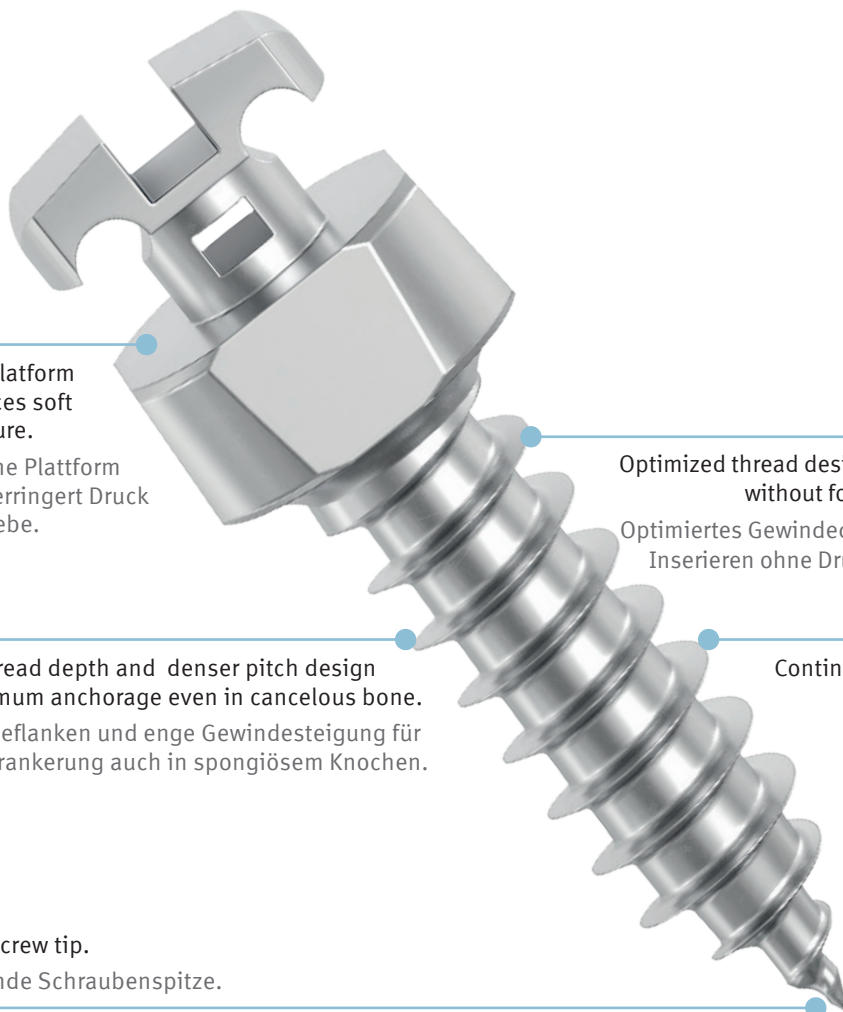
Optimized thread design leads to easy insertion without force (cork screw principal).

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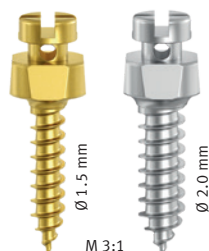
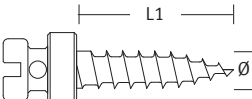
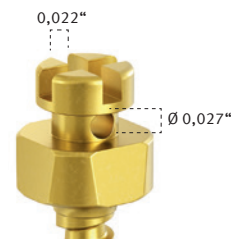
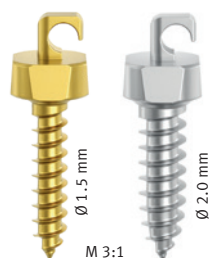
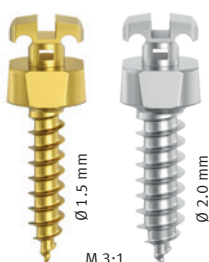
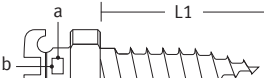
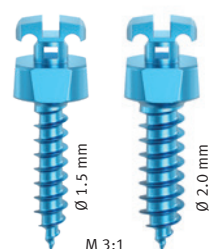
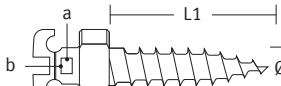
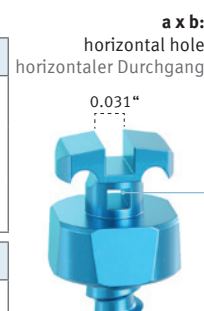
Illustrations are for illustrative purposes only and are not to scale, except where scale is indicated.

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
Screwholder for contra-angled handpiece

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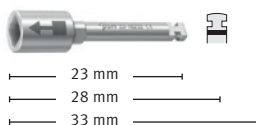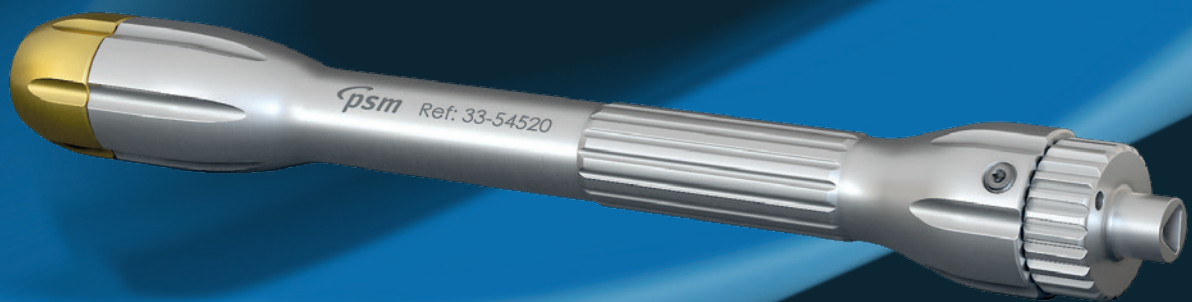
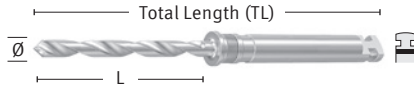
The instruments can be used for several **PSM Medical** products. In order to find the right instruments for your use, the sterile screwholders are matching in colors with the correct drill.

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The extension of 16 mm works for all rotary instruments of the **quattro®** and **BENEFIT®** System with a latch type DENTAL attachment (max. rpm 600).

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
Drill stop | Bohrerstopp


The drill stop prevents the clinician from unintentionally drilling too deep. They assure an accurate working length from 7 to 13 mm in 2 mm increments. By sliding the stop sleeve over the appropriate **PSM** drill, the working length is adjusted to the desired depth. Removal of the sleeve is easy by using the buffer.

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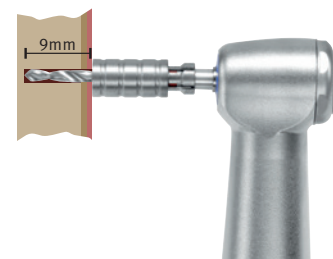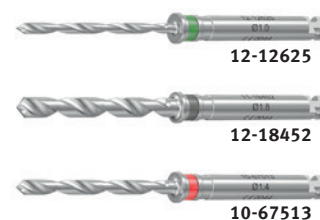
Screwholder | Schraubenhalter

All **quattro®** screwholders are marked with an arrow indicating the slot, hole or hook position on the **quattro®** screw during insertion.

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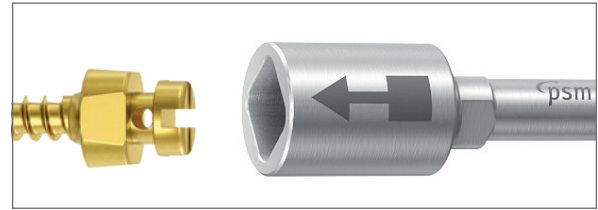
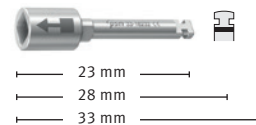
Screwdriver | Schraubendreher

The manually turned unit is connected to your existing contra-angled handpiece (blue 1:1) that enables its use like an angled screwdriver.

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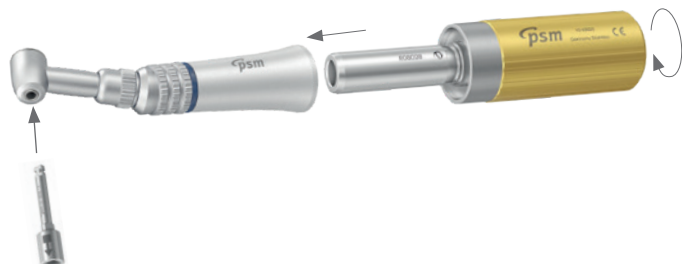Manually turned unit for contra-angled handpieces

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Manually turned unit mod. to Pauls, with adjustable torque from 0 – 40 Ncm

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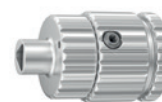
Battery Screwdriver – iSD900 | Akku Schraubendreher iSD900

Cordless Battery Screwdriver with Torque Calibration System for the controlled fixation of PSM Miniscrews.

The iSD 900 will guarantee precise torque control due to the unique – easy to use – torque calibration system (10 – 40 Ncm). Time-saving and precise, riskfree insertion and tightening of screws and abutments. The complete set contains a reliable and user friendly handpiece for Dental attachments. Reverse mode is indicated with sound. Standard rechargeable AAA Ni-Mh batteries can be used.

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
Starter Set | Starter Set

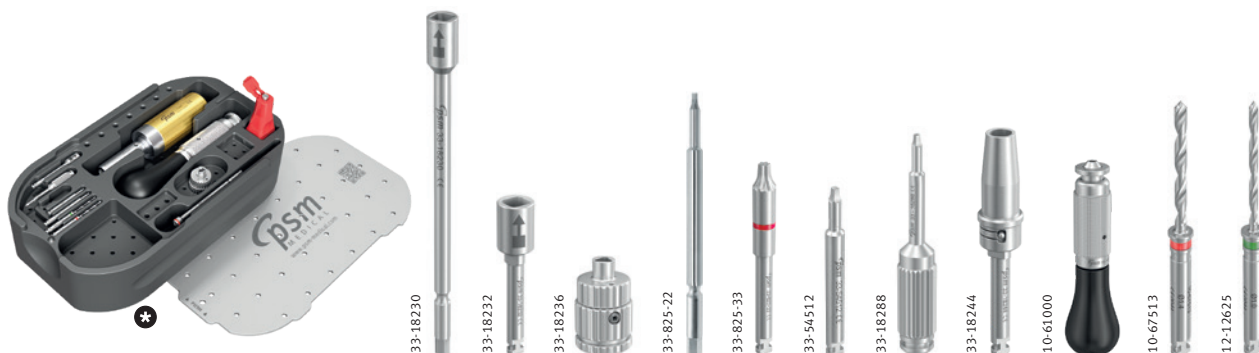quattro® / BENEFIT® STARTER SET

33-11500	quattro® / BENEFit® INSTRUMENTS SET		without implants
consisting of: bestehend aus:			
	Storage	Aufbewahrung	
73-31960	quattro® / BENEFit® tray, empty	quattro® / BENEFit® Behälter, leer	
	Instruments	Instrumente	
33-18230	Screwholder QB*, long, for 10-61000	Schraubenhalter QB*, lang, für 10-61000	
33-18232	Screwholder QB*, short, 23 mm, Dental, for contra-angled handpieces	Schraubenhalter QB*, kurz, 23 mm, Dental für Winkelhandstück	
33-18236	Thumb screw for Dental mandrel	Handgriff für Instrumente mit Dentalansatz	
33-825-22	BENEFit® DIRECT TX Screwholder for 10-61000	BENEFit® DIRECT TX Schraubenhalter, für 10-61000	
33-825-33	BENEFit® DIRECT TX Screwholder, short	BENEFit® DIRECT TX Schraubenhalter, kurz	
33-54512	BENEFit® Screwholder for Abutment, 20 mm	BENEFit® Eindrehinstrument für Abutment, 20 mm	
33-18288	Laboratory Screwdriver for Abutments etc.	Labor Schraubendreher für Abutments etc.	
33-18244	Drill extension +16 mm, DENTAL	Bohrer Verlängerung +16 mm, DENTAL	
10-61000	Screwdriver handle	Schraubendreher Handgriff	
10-67513	Drill, 1,4 x 33 mm, WL 15 mm, DENTAL, red	Bohrer, 1,4 x 33 mm, AT 15 mm, DENTAL, rot	
12-12625	Drill, 1,0 x 33 mm, WL 15 mm, DENTAL, green	Bohrer, 1,0 x 33 mm, AT 15 mm, DENTAL, grün	

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*QB: quattro® & BENEFIT®

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quattro® / BENEFIT® tray for instruments and implants, without content, with lid

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73-31960

quattro® / mentoplate® tray for instruments, without content, with lid

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*QB: quattro® & BENEFIT®

BENEFIT®

The instruments can be used for several **PSM Medical** products. In order to find the right instruments for your use, they are indicated in color.

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quattro®.mini

 Video

PSM –
The Original quattro®
TAD System

Miscellaneous Sonstiges

Sterile Packaging | Sterile Verpackungen

The instruments can be used for several **PSM Medical** products. In order to find the right instruments for your use, the sterile screwdrivers are matching in colors with the correct drill.

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 **BENEFIT®**
 **smartlock®**
 **quattro®**
 **mentoplate®**
 **quattro®mini**

Implants – in single sterile packaging

The outer and inner packaging contains labels indicating UDI code-like article number, LOT number, expiration date, etc. for the patient folder and for full traceability. The sterile screws are contained in a small colour-coded screw carrier which protects the screw and at the same time acts as a pick up tray. This carrier is placed in a clear, sealed blister package. The screw can be removed easily from the carrier by means of the self retaining screwdriver.

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Direct screw removal from the sterile holder.

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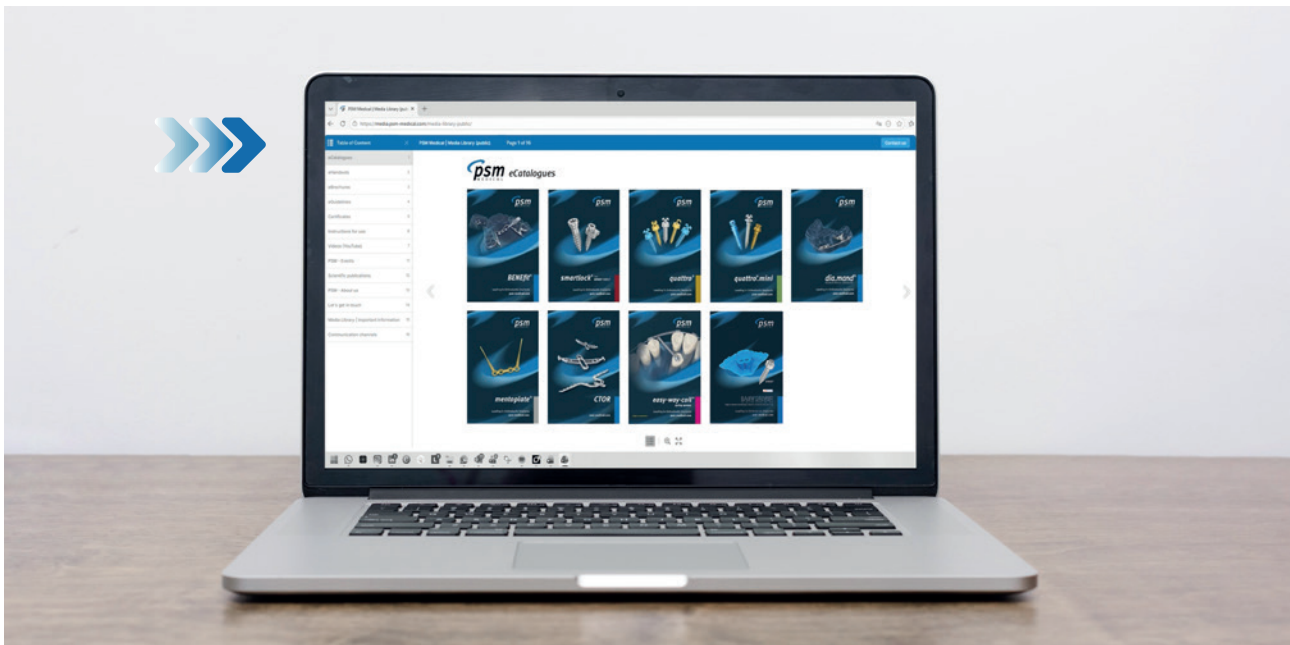
PSM Digital Media Library – Your Real-Time Source for Orthodontic Excellence

Welcome to the official **PSM Medical Media Library**.

This central hub provides clinicians, distributors, partners and interested parties with instant, 24/7 self-service access to our complete portfolio.

- Always Up-to-Date: Our online documents (ePublications) are maintained in real-time.
- Verified Content: Every publication is officially approved by PSM Management, ensuring verified and revision-controlled information.
- Smart Access: No special software is needed. Access Catalogues, Guidelines, Certificates, Videos, Scientific Publications etc. on any device via a dedicated URL.
- Responsibility Note: We provide a PDF download option for offline use. However, please note that only the online version is part of our automated revision service. To ensure you are using the latest clinical data, we recommend referencing the live link.

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**Maximum Anchorage.
Minimal Design.**

**Maximale Verankerung.
Minimales Design.**

In addition to the proven PSM thread design, the **quattro®.mini** screw introduces an even smaller bracket head design that revolutionises patient comfort based on the proven **quattro®** design.

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quattro® RH micro

quattro® Aligner

RH: Round head design | Runder Kopf Design
BH: Bracket head design | Bracket Kopf Design

Developed in cooperation with: | Entwickelt in Zusammenarbeit mit:
Dr. Nicola Derton Studio Odontoiartico Specialistico, Conegliano (TV), Italy

One Screw – Many Options

The quattro® system offers versatile screws with a standardised thread design (1.3 - 2.3 mm) for easy insertion and reliable treatment success. The new quattro® MINI screw is designed more compactly to adapt to different anatomical conditions, while retaining the proven self-drilling thread for high stability and low loss rate. Extremely sharp thread flanks and optimised thread pitch make it the ideal complement for mini-anchor screws.

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See our **quattro®.mini** catalogue

Specifications and design are subject to change without prior notice. All pictures, photos and product descriptions are the property of PSM Medical GmbH. Utilization and copies by third parties have to be authorized by PSM Medical GmbH. All rights reserved! All trademarks and brands mentioned in this brochure are subject to unrestricted regulations of the trade mark laws valid and right of ownership of the registered owner.

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PSM North America, Inc.